FERMACAPO – UNIVERSAL HEAD IMMOBILIZER

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ITALIANO

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ENGLISH

INDICATIONS : FERMACAPO is designed to aid immobilization of a patient's head and neck during transportation when injury is suspected. The head immobilizer can be used on Spinal Board (BAR025) NFC System foldable stretchers (BAR 016- BAR 021) foldable in two (BAR 022)and SCOOP stretcher (BAR 017) . Minimal interference with X-ray

SPECIFICATIONS : FERMACAPO is compound of a Base plate fixed on the stretcher with two + one side retentionstraps secured with the VELCRO ; two Support pads to immobilize patient's head and two Strap pads to fix foreheadand chin (see fig. 1). FERMACAPO is realized with a soft material covered with a special washable vinyl paint

INSTRUCTIONS : secure the head immobilizaer Base plate to the stretcher (see fig. 2). Using EMS transfer techniques, move the patient onto the stretcher ; place the straight side of each Support pad against each side of the patient's head (see fig 3) and place forehead/chin straps over the patient securing with VELCRO (see fig. 4). In cases of suspected head and/or neck injury a cervical collar should be applied prior to moving the patient.

CAUTION : should be applied by skilled staff

CLEANING AND MAINTENANCE INSTRUCTIONS : after using verify integrity of FERMACAPO ; eventually replace damaged parts. Clean the head immobilizer with warm water and soap

STORAGE : should be kept in a dry room, dust free . Storage as a compact unit .

It is necessary to report any serious accident occurring in relation to the DM provided by PVS SPA to the Manufacturer and the competent authorities

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